

Instruction Manual

Affinity Marlin

AFALU4/ OR AFALU428



Specifications

Affinity Marlin

Product Code: AFALU4 or AFALU428



Height Adjustment:	24.4"-32.5" (62cm-82.5cm)
Weight:	25" - 14.5kgs (32lbs) 28" - 16.5kg (36.3lbs)
Dimensions:	Length 73" (185cm), Width 25" (63cm)/ Width 28" (71.2cm)
Materials:	Aluminium frame and base with 2" semi firm, sports, Therafoam padding, PVC vinyl (navy couches), PU (white couches).
Weight Capacity:	450lbs (205kgs/32stones of impact)
Warranty:	2 years
Face Cradle/Paper Towel Holder Ports:	Both ends
Carry Case:	Included
Colour:	White Navy
Shipping W&Ds (approx):	25" - L96cm x W23cm x H70cm. Gross weight: 19kgs. 28" - L96cm x W24cm x H78cm. Gross weight: 21kgs

Instructions

Setting up the Couch:



With the couch set on its rubber bumpers, undo the central metal securing clip.



Gently open out the couch, checking to make sure none of the cables are trapped around the legs. Forcing the legs to open with trapped cables can damage your couch.



Once couch is fully open, holding the carry handles gently ease the couch upright.



With one hand placed on the middle crease, gently lift one end of the couch to ensure the cables are fully tense.



Adjusting the Couch Height:



Push the button in and gently pull the leg extender down. Repeat on all four legs.



Instructions

How to Attach the Face Cradle and Arm Rest:

This is **not** included with the Affinity Marlin as standard but is an optional extra (code AFPWR).




Face cradle cushion



Face Cradle frame



Hanging arm rest

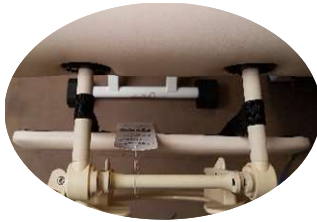

Slide the face cradle into the port holes on the couch.



Lock the face cradle into position using the lever.



To change the angle of the face cradle frame, simply release the locking lever, move the cradle into desired position and lock the lever.




How to Take Down Your Massage Couch:

Lower the couch on to its side, resting on the rubber bumpers.



Gently apply pressure at the elbow point, pushing (see below yellow circle) towards the couch base. This will cause the legs to fold in and the couch will start to close.

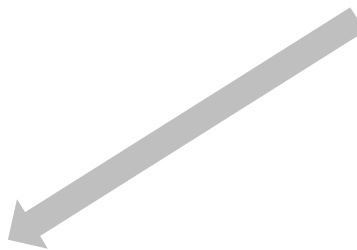

Fold the legs into the table, taking care to ensure all cables are tucked inside.



Continue to fold the couch closed and lock shut with the central clip.



How to Care for Your Upholstery:

In an effort to safeguard your product's quality and maintain the life of your couch, we advise the following:

- Avoid storing your table in extreme cold temperatures as the cold will cause cracking.
- Avoid storing your table in extreme hot temperatures. Heat will cause the upholstery to stretch and pucker. Keep your table away from heaters, electrical appliances and out of direct sunlight.
- When in use, always protect your couch with accessories such as covers and sheets.
- Clean your upholstery daily, taking care to clean inside the breathe hole and the carry handles as even the oils naturally found on our fingers can cause upholstery to break down. Regular cleaning will help to protect it against oils, stains and dirt, which, if left, will cause cracking and permanent stains.
- To clean the upholstery, we suggest a mild mixture of anti bacterial detergent (avoiding those containing citric acid) and water in a spray bottle. Spray the table with the solution and wipe down the table with a clean, damp towel.
- Where possible, try to use both ends of your couch to avoid uneven use.



The use of alcohol will dry out the vinyl and cause it to crack.
Prolonged exposure to extreme temperatures may cause damage to vinyl.

General Maintenance:

Regularly check all bolts and screws and tighten if necessary.

Check wires and cables for signs of wear or splitting.



Thank you for choosing Affinity.

Every care has been taken to ensure that this product reaches you in excellent working condition.

Should you have any questions or concerns please refer back to your original retailer who will be happy to assist.